



CORFLEX®

Designed for life.

Universal Wrist Lacer Orthosis

APPLICATION INSTRUCTIONS

Fitting and Application:

1. Open both wrist and thumb straps. Slide hand into the splint making sure palmar stay lines up below palmar crease.



2. Align lacing stay assembly on the back of the hand with the middle two knuckles, making sure the assembly does not extend beyond the brace on either end.



3. Secure the narrow strap located between the thumb and index finger along the back of the splint. Cinch the wider lacing strap to desired comfort and wrap around the splint. Attach hook closure to loop.



4. Re-adjust closures to desired tension for a comfortable fit. Trim any excess strap length if desired.



Care instructions:

The Universal Wrist Lacer Orthosis may be hand washed using warm water and detergent for delicate fabrics. Remove metal stays where possible. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer.

Caution:

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If any additional pain or symptoms occur while using this device, seek medical attention.

Indications:

Indicated for Carpal Tunnel Syndrome, tendonitis, sprains or strains, soft tissue injury, wrist contracture and post-cast removal.

Contraindications:

Do not use on fractures; do not for use on patients with circulatory or sensation issues.

Warranty:

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to mis-use or application the warranty will be voided at the discretion of Corflex.

To Reorder:

Product Number

Right	Left
73-7500	73-7501



Sizing: One size fits patients with a wrist circumference of 5" to 9".