



CORFLEX®

Designed for life.

CORFLEX.COM | 800 426 7353

Lower Leg & Ankle Non-Pneumatic Walking Splint Application Instructions

Fitting and Application:

1. Release straps from liner and remove from metal uprights. Place injured foot into liner and fit heel up against back of liner. Close forefoot cover over the foot. Secure the upper calf liner with hook and loop closure. Insert optional ankle pads now if desired.



2. Place the foot with liner into walking splint. Thread the strap closest to the toes through plastic buckle and secure onto itself. Repeat this process for all remaining straps working your way towards the knee. Center the uprights in line with the leg and remove plastic sleeves and engage the hook to body of the liner.



3. Test for desired comfort level, do not overtighten straps.



CORFLEX®

Designed for life.

CORFLEX.COM | 800 426 7353

Lower Leg & Ankle Non-Pneumatic Walking Splint Application Instructions

Fitting and Application:

1. Release straps from liner and remove from metal uprights. Place injured foot into liner and fit heel snugly in posterior section. Close forefoot cover over the foot. Secure the upper calf liner with hook and loop closure. Insert optional malleolus pads now if desired.



2. Place the foot with liner into walking splint. Thread the strap closest to the toes through plastic buckle and secure onto itself. Repeat this process for all remaining straps working your way towards the knee. Center the uprights in line with the leg and remove plastic sleeves and engage the hook to body of the liner.



3. Test for desired comfort level, do not overtighten straps.



Care Instructions

The Corflex Non-Pneumatic Walking Splint liner may be hand washed using warm water and detergent for delicate fabrics. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer. Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

Caution

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If additional pain or symptoms occur while using this device, seek medical attention.

Indications

For post injury or post-operative application to grade 1, 2, or 3 ankle sprains, and stable or internally fixed fractures of the foot or ankle.

Contraindications

For unstable fractures or for fractures of the proximal tibia or fibula.

Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex.

To Reorder

Non-Pneumatic Walking

	XS	S	M	L	XL
Lower Leg	79-1049	79-1050	79-1051	79-1052	79-1053
Ankle	79-1059	79-1060	79-1061	79-1062	79-1063

Non-Pneumatic Walking Splints Replacemant Liners

	XS	S	M	L	XL
Lower Leg	79-1149	79-1150	79-1151	79-1152	79-1153
Ankle	79-1159	79-1160	79-1161	79-1162	79-1163

Sizing Guide	Mens	Women
X-Small	4 to 4 ½	3 to 5
Small	5 to 6	5 ½ to 7
Medium	6 ½ to 10	7 ½ to 11
Large	10 ½ to 12	11 ½ to 13
X-Large	12 ½+	14+

Measured by Shoe Size.



Care Instructions

The Corflex Non-Pneumatic Walking Splint liner may be hand washed using warm water and detergent for delicate fabrics. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer. Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

Caution

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If additional pain or symptoms occur while using this device, seek medical attention.

Indications

For post injury or post-operative application to grade 1, 2, or 3 ankle sprains, and stable or internally fixed fractures of the foot or ankle.

Contraindications

For unstable fractures or for fractures of the proximal tibia or fibula.

Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex.

To Reorder

Non-Pneumatic Walking

	XS	S	M	L	XL
Lower Leg	79-1049	79-1050	79-1051	79-1052	79-1053
Ankle	79-1059	79-1060	79-1061	79-1062	79-1063

Non-Pneumatic Walking Splints Replacemant Liners

	XS	S	M	L	XL
Lower Leg	79-1149	79-1150	79-1151	79-1152	79-1153
Ankle	79-1159	79-1160	79-1161	79-1162	79-1163

Sizing Guide	Mens	Women
X-Small	4 to 4 ½	3 to 5
Small	5 to 6	5 ½ to 7
Medium	6 ½ to 10	7 ½ to 11
Large	10 ½ to 12	11 ½ to 13
X-Large	12 ½+	14+

Measured by Shoe Size.



669 EAST INDUSTRIAL PARK DRIVE
MANCHESTER, NH 03109
CORFLEX.COM | TOLL FREE: 800 426 7353



669 EAST INDUSTRIAL PARK DRIVE
MANCHESTER, NH 03109
CORFLEX.COM | TOLL FREE: 800 426 7353