



CORFLEX®

Designed for life.

CORFLEX.COM | 800 426 7353

Marathon™ Air Walker Ankle

Application Instructions



The Marathon Air Walker features an inflatable interior bladder that allows for a custom fit while providing compression and stabilization throughout recovery.

Fitting and Application

1. Place leg inside liner and position heel as far back as possible. Close forefoot cover over top of foot and fasten it snugly.

POSITION FOREFOOT COVER OVER TOP OF FOOT



2. Wrap liner around the front of lower leg and secure with hook and loop closures.

WRAP LINER AROUND FRONT/LOWER LEG

FOREFOOT COVER



Fitting and Application (continued)

3. Situate front shell over liner taking care to properly position it over the inflation bulb.

Check that the inflate/deflate knob is in the DEFLATE position before tightening straps, this will allow air to escape from the bladder.

The front shell should not contact the foot or leg, adjust as needed.



4. Insert initial ankle strap through plastic buckle and secure strap. Always secure this strap first to encourage proper positioning in the walker.

SECURE ANKLE STRAP FIRST FOR PROPER FIT



5. Secure forefoot strap. Insert top strap through outer shell slot, then through front shell and fasten back on itself.

INSERT STRAP THROUGH REAR AND THEN FRONT OUTER SHELL



6. Test for desired comfort level, do not overtighten straps.

CAUTION: DO NOT OVERTIGHTEN STRAPS

TIGHTEN STRAPS UNTIL SECURE AND COMFORTABLE



7. To inflate liner, turn the inflate/deflate knob clockwise until fully opened and press and release inflation bulb until liner is providing support and comfort.

INFLATE LINER CAUTION: DO NOT OVERINFLATE LINER


PRESS AND RELEASE INFLATION BULB AND TURN THE INFLATE/DEFLATE KNOB COUNTERCLOCKWISE



Fitting and Application (continued)

7. To release air from liner, turn the inflate/deflate knob counterclockwise several times. If at any time the liner feels too tight, immediately release air by turning the inflate/deflate knob counterclockwise.

TURN THE INFLATE/DEFLATE KNOB COUNTERCLOCKWISE TO RELEASE AIR

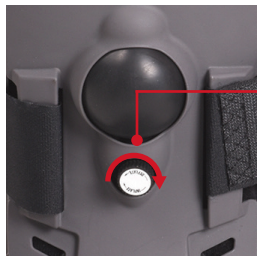

CLOCKWISE INFLATES LINER



COUNTERCLOCKWISE RELEASES AIR FROM THE LINER

- DEFLATE LINER BEFORE REMOVING THE AIR WALKER.
- PRESSURE CAN BE ADJUSTED BY TIGHTENING AND LOOSENING STRAPS.
- DO NOT OVERINFLATE.
- FOLLOW PHYSICIANS INSTRUCTIONS.

Marathon Air Walkers

Designed to increase stability and decrease edema, our walker features a built-in pump to achieve desired level of compression. Ideal for stable foot/ankle fractures, severe sprains, soft tissue injuries, post cast removal and post surgical application. Custom sole and low profile non-skid bottom and adjustable toe protector promotes natural gait. Available in two heights, ankle and tall.



Troubleshooting:

If you cannot inflate or deflate the liner check that metal valve located on the top of the inflation bulb is not blocked.

If you cannot inflate or deflate the liner check that the inflate/deflate valve is in the correct position.

If boot does not fit snugly at top of boot, move the top strap to the alternate vertical top slot located in the plastic shell.

Care Instructions:

Liner should be hand washed only using warm water and detergent for delicate fabrics, repeatedly rinsed and line dried only. Do not use bleach or place in a dryer. Wipe plastic shell with damp cloth, you may use disinfectant or mild soap and wipe dry.

Caution:

This product has been designed and tested based on single patient use only. It should never be used by multiple patients. If additional pain, swelling or symptoms occur while using the device, seek medical attention. Product should be applied by a licensed or certified healthcare practitioner and as prescribed by a physician or other qualified medical authority.

Indications:

Indicated for stable foot fractures, severe sprains, soft tissue injuries, post-op and post-cast removal.

Warranty:

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex.

To Reorder:

Product Number

	XS	S	M	L	XL
Marathon Air Walker Ankle	79-2020	79-2021	79-2022	79-2023	79-2024
Replacement Liner Ankle	79-2040	79-2041	79-2042	79-2043	79-2044

Sizing Guide

	XS	S	M	L	XL
Men's Size	3-4	4½-7	7½-10	10½-12½	13+
Women's Size	4-5½	6-8	8½-11	11½-13	14+

Measured by Shoe Size



CORFLEX
Designed for life.

CORFLEX | 669 EAST INDUSTRIAL PARK DRIVE | MANCHESTER, NH 03109
CORFLEX.COM | TOLL FREE: 800 426 7353 | FAX: 603 623 4111