



CORFLEX®

Designed for life.

CORFLEX.COM | 800 426 7353

ReAlign™ Spinal Orthosis Extension Kit LO, LSO and LSO PLUS Versions Application Instructions

Fitting and Application:

Extension Kit is designed and tested for use only with the ReAlign Spinal Orthosis (sold separately). Use the Extension Kit when fitting patient's with a waist/hip circumference from 50" to 75".

1. Open the brace and lay flat with the inside surface facing up. Remove grey pull tabs from the belt and place off to the side. Remove side panels (LSO Plus model only) and set aside.



2. Release hook and loop closure at end of belt and slide belt end out through large D-Ring. Angle end of belt for easier removal.



3. With the inside of the extension panel facing up, feed the label end through large D-Ring. Push back the edges of the surrounding fabric and thread through at an angle for easier assembly.



4. Fold over and secure end of belt near the large D-Ring. Overlap opposite end of extension panel with original belt.



5. Adjustments to brace circumference using the extension kit are best made where the panel overlaps the belt. Apply belt and make adjustments at the overlap equally on both sides until proper fit is achieved. Reattach side panels as needed (LSO Plus model only).



Care Instructions

The ReAlign Extension panels may be hand washed using warm water and detergent for delicate fabrics. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer.

Caution

This product has been designed and tested based on single patient use. Since it is a single use device it should never be used by multiple patients. If any additional pain or symptoms occur while using this device, seek medical attention.

Indications

Indicated for acute and chronic low back pain, disc herniation, stable, non-displaced fractures and post-op stabilization.

Contraindications

Contraindicated for unstable, displaced fractures, women who are pregnant, persons with circulatory, pulmonary, cardiovascular or other conditions where increased abdominal pressure could worsen risks..

Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex.

To Reorder

Product

ReAlign Extension Kit - 16" : 35-3912 :



CORFLEX
Designed for life.

669 EAST INDUSTRIAL PARK DRIVE
MANCHESTER, NH 03109
CORFLEX.COM | TOLL FREE: 800 426 7353