



CORFLEX®

Designed for life.

CORFLEXGLOBAL.COM | 800 426 7353

CoolTex™ Ag 13" Patella Stabilizer with Cor-Trak Buttress and Hinge

Application Instruction

Fitting and Application:

1. Unfasten and loosen both the calf and thigh straps. If desired remove U-shaped Cor-Trak patella buttress and position where support is needed.



2. Grasp tabs at wider end of stabilizer noting seam positioned to back of leg. Insert foot into sleeve and pull onto leg.



3. Position sleeve so that stays are parallel to each other on the side of the leg and the patella opening is centered on the kneecap.



4. Buttress should be in contact with kneecap to provide support. Fasten the support straps at the thigh and calf for comfortable fit without inhibiting circulation.



Care Instructions

The CoolTex Ag Patella Stabilizer with Cor-Trak Buttress and Hinge may be hand washed using warm water and detergent for delicate fabrics. Remove hinges prior to washing. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer.

Caution

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If additional pain or symptoms occur while using this device, seek medical attention.

Indications

Patella tracking abnormalities, tendonitis, knee strains or sprains, sports injuries, post-op rehabilitation and arthritis.

Contraindications

Not for use on patients with circulatory or sensation issues. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use and consult your medical professional.

Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex Global.

To Reorder

Product Number		Size	Measurement		
Open Popiteal	Closed Popiteal		6" Above Mid-Pat	Mid-Pat	6" Below Mid-Pat
85-5072	85-5082	XS	16"-18"	12"-13"	11"-12"
85-5073	85-5083	S	17"-19"	13"-14"	12"-13"
85-5074	85-5084	M	18"-20"	14"-15"	13"-14"
85-5075	85-5085	L	19"-21"	15"-16"	14"-15"
85-5076	85-5086	XL	20"-22"	16"-18"	15"-16"
85-5077	85-5087	2XL	21"-23"	18"-20"	16"-17"
85-5078	85-5088	3XL	23"-26"	20"-22"	17"-18"
85-5079	85-5089	4XL	26"-29"	22"-24"	18"-19"



CORFLEX
GLOBAL

669 EAST INDUSTRIAL PARK DRIVE
MANCHESTER, NH 03109
CORFLEXGLOBAL.COM | 800 426 7353