



CORFLEX®

Designed for life.

CORFLEXGLOBAL.COM | 800 426 7353

CoolTex™ Ag 8" Thumb Spica with Strap Application Instruction

Fitting and Application:

1. Release all hook and loop closures. If desired, the removable aluminum palmar, dorsal and thumb stays may be shaped to fit the curvature of the hand.
2. Slide hand into splint so wide stay fits comfortably in palm. Thumb IP joint should extend past opening of thumb tunnel. Beginning at the distal strap, close both hook and loop closures to desired tension for a comfortable fit.
3. Wrap thumb strap and adjust for a comfortable fit. Trim any excess strap length from all straps as needed.
4. Lastly, wrap the wide elastic strap and secure to loop. Strap should fit snug and provide support but not restrict circulation.



Care Instructions

The CoolTex Ag Thumb Spica with Strap may be hand washed using warm water and detergent for delicate fabrics. Remove stays prior to washing. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer.

Caution

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If additional pain or symptoms occur while using this device, seek medical attention.

Indications

deQuervain's Syndrome, tendonitis, thumb strains or sprains and post-cast removal.

Contraindications

Do not use on fractures. Not for use on patients with circulatory or sensation issues. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use and consult your medical professional.

Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex Global.

To Reorder

Product Number		Size	Measurement (Wrist Circumference)
Right	Left		
74-8600	74-8610	XS	4"-5"
74-8601	74-8611	S	5"-6"
74-8602	74-8612	M	6"-7"
74-8603	74-8613	L	7"-8"
74-8604	74-8614	XL	8"-9"
74-8605	74-8615	2XL	9"-10½"
74-8606	74-8616	3XL	10½"-12"



669 EAST INDUSTRIAL PARK DRIVE
MANCHESTER, NH 03109
CORFLEXGLOBAL.COM | 800 426 7353