



CORFLEX®

Designed for life.

CORFLEXGLOBAL.COM | 800 426 7353

CoolTex™ Ag 9" Wrist Splint with Strap Application Instruction

Fitting and Application:

1. Release all hook and loop closures. If desired, the removable aluminum palmar, and dorsal stays may be shaped to fit the curvature of the hand.



2. Slide hand into splint so wide stay fits comfortably in palm. Beginning at the distal strap, close both hook and loop closures to desired tension for a comfortable fit.



3. Lastly, wrap the wide elastic strap and secure to loop. Strap should fit snug and provide support but not restrict circulation.



Care Instructions

The CoolTex Ag Wrist Splint with Strap may be hand washed using warm water and detergent for delicate fabrics. Remove stays prior to washing. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer.

Caution

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If additional pain or symptoms occur while using this device, seek medical attention.

Indications

Carpal Tunnel Syndrome, tendonitis, wrist strains or sprains and post-cast removal.

Contraindications

Do not use on fractures. Not for use on patients with circulatory or sensation issues. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use and consult your medical professional.

Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex Global.

To Reorder

Product Number		Size	Measurement (Wrist Circumference)
Right	Left		
74-7600	74-7610	XS	4"-5"
74-7601	74-7611	S	5"-6"
74-7602	74-7612	M	6"-7"
74-7603	74-7613	L	7"-8"
74-7604	74-7614	XL	8"-9"
74-7605	74-7615	2XL	9"-10½"
74-7606	74-7616	3XL	10½"-12"



669 EAST INDUSTRIAL PARK DRIVE
MANCHESTER, NH 03109
CORFLEXGLOBAL.COM | 800 426 7353