

Recovery™ Post-Op Knee Brace

Rebuild strength and confidence with an affordable, tailor made solution for post-op knee rehabilitation.



CORFLEX®

Designed for life.

CORFLEXGLOBAL.COM



The Recovery Post-Op Knee brace is a robust, cost-effective solution that provides all the features needed for post-operative care. Universal sizing accommodates a wide variety of patients and the color-coded dial has easy-to-use ROM adjustments. Eight drop lock settings allow for quick immobilization. The telescoping uprights can be contoured for an individualized patient fit.

ADDITIONAL FEATURES:

- Available in Lite or Full Foam versions, the Recovery Post-Op Knee Brace offers telescoping uprights and secure quick-release buttons to prevent unwanted adjustments.
- Indicated for range of motion control following ACL, PCL, MCL and LCL surgeries, meniscal repairs, patella realignment, regenerative chondroplasty, stable femoral fractures, total knee replacements and tibial plateau fractures.
- Custom-bendable aluminum uprights allow for contouring; latex free foam liner.

PRODUCT NUMBER

UNIV

Lite

75-7560

Full Foam

75-7561

Sizing Guide

Fits up to 34" thigh circumference

Length Adjustable 18"-26"


Measurement taken from leg circumference 6" above mid patella.

Suggested Billing Codes

Knee Orthosis, Adjustable Knee Joint

L-1832 (Custom-Fit) or L-1833 (OTS)

NOT ALL BODIES ARE THE SAME A POST-OP CUSTOM BRACING SOLUTION

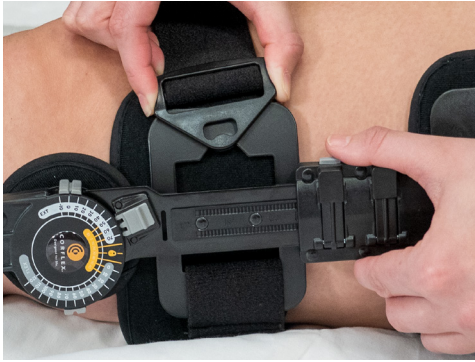

Micro Adjust System

Overall length adjusts from 18" to 26" in 1/16" increments with dual locks for secure engagement



Proportional Ankle Cuff

Designed for a secure fit above the ankle, especially on smaller circumferences to prevent migration



Drop Lock Slide Button

Eight locking positions to quickly immobilize the brace from -10° to 60°



Metal Uprights

Custom bendable, aluminum uprights are easily contoured without tools to help minimize migration



Flexion & Extension Adjustments

Pull tabs adjust range of motion from -10° to 120° in 10° increments



Trim-to-Fit Straps

Simply cut to length and apply alligator-style hook to ends



CORFLEX GLOBAL | 669 EAST INDUSTRIAL PARK DRIVE | MANCHESTER, NH 03109
CORFLEXGLOBAL.COM | TOLL FREE: 800 426 7353 | FAX: 603 623 4111